

**MARSYS INC.**  
**TYPE P650 PACKING INSTRUCTIONS FOR ROAD TRANSPORTATION ONLY**  
**(NON-FLIGHT)**

THIS MARSYS Clinical Specimen Carrier/Transporter has been designed and fabricated with care by MARSYS in recognition of the fragile nature of the items for which it is intended to transport. However, because of the fragile nature of those items and because MARSYS has no control over the way that Carrier/Transporter will be assembled, filled or handled, MARSYS will not be responsible to any party for the loss of its contents or for any damage to persons or property resulting from the use or misuse of the Clinical Specimen Carrier/Transporter.

**INSTRUCTIONS:**

Secure carrier/transporter in upright position, as per label/arrow direction.

Insert hard body or ziplock waterproof secondary inner liner into transporter.

Insert appropriate absorbent inserts (see inserts below) for the protective packaging and absorbent requirements of the sample specimens or contents intended for shipment.

Place specimens into insert openings, as may apply.

Insert applicable documentation into document pouch, if provided, and place in unit.

Close carrier/transporter lid and secure lid closure mechanism, seal as required.

Insert all other shipping/labeling documentation into document pouch, exterior, if provided, as may be required through regulation, law and/or company policy.

**IT IS A REQUIREMENT OF THE MARSYS CLINICAL SPECIMEN CARRIER/TRANSPORTER SYSTEM THAT ALL VOIDS WITHIN THE CARRIER/TRANSPORTER BE FILLED WITH INSERTS, SEE BELOW. THIS ENSURES THE SAFE AND SECURE TRANSPORTATION OF THE CONTENTS.**

**THE MARSYS CLINICAL SPECIMEN CARRIER/TRANSPORTER MODELS/SYSTEMS ARE DESIGNED TO TRANSPORT/SHIP TRANSPORT CANADA TYPE P650 PACKAGINGS ONLY.**

**MISUSE SHALL AT THE SOLE RISK OF THE SHIPPER.**

**INSERTS**

The following MARSYS inserts are available. ANY COMBINATION of these inserts may be used to meet the above stated requirement:

VACUTAINER INSERT - Model MVB/21-SN Will hold 21 vacutainers, 3.5-10ml.

URINE/OTHER INSERT - Model MU2-SN Will hold containers, 40 - 90ml.

HOLLOW HOLDER INSERT - Model MHH34-SN.

Hollow Holder Inserts will also hold ice packs when specimen integrity is required.

**NOTICE**

MARSYS SN SERIES UNITS ARE NOT DESIGNED FOR USE WITH ICE CUBES - USE ONLY  
ICE PACKS OR DRY ICE.

**CLEANING**

HAND WIPE ONLY AND/OR IN ACCORDANCE WITH USER SAFETY/TECHNICAL  
INSTRUCTIONS FOR SAME.